Alcotest[®] 7410^{Plus}
Alcotest[®] 7410^{Plus} RS

Atemalkohol-Messgerät
Breath Alcohol Monitor

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For Your Safety

Strictly follow the Instructions for Use

Any use of the instrument requires the full understanding and strict observation of these instructions. The instrument is only to be used for the purposes specified here.

Maintenance

Repair of the instrument may only be carried out by trained personnel.
We recommend that all repairs be carried out by DrägerService.
Only original Dräger spare parts may be used for maintenance.
Observe chapter "Maintenance Intervals".

Accessories

Do not use accessory parts other than those in the order list.

Not for use in areas of explosion hazard

Alcotest[®] 7410^{Plus} / Alcotest[®] 7410^{Plus} RS is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

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Dräger Safety AG & Co. KGaA

Liability for proper function or damage

The liability for the proper function of the instrument is irrevocably transferred to the owner or operator to the extent that the instrument is serviced or repaired by personnel not employed or authorized by DrägerService or if the instrument is used in a manner not conforming to its intended use.

Dräger cannot be held responsible for damage caused by non-compliance with the recommendation given above. The warranty and liability provisions of the terms of sale and delivery of Dräger are likewise not modified by the recommendations given above.

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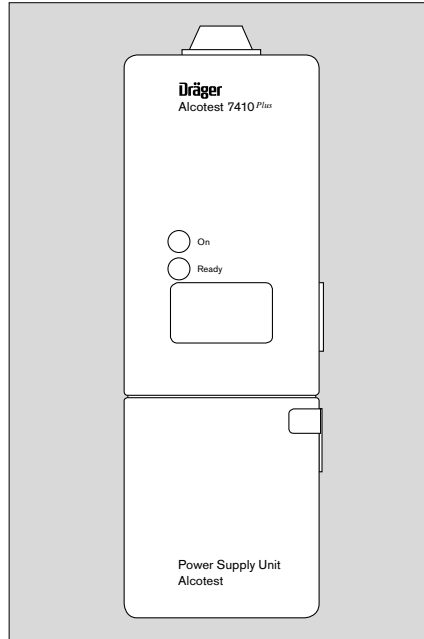
Intended Use

For rapid determination of alcoholic intoxication by measuring breath alcohol concentration.

For mobile use at test sites or for stationary use, also in combination with the Alcotest® 7410 Printer.

Alcotest® 7410^{Plus} / Alcotest® 7410^{Plus} RS stores, depending on version, up to 9700 test results. Additionally elementary statistics are available at once.

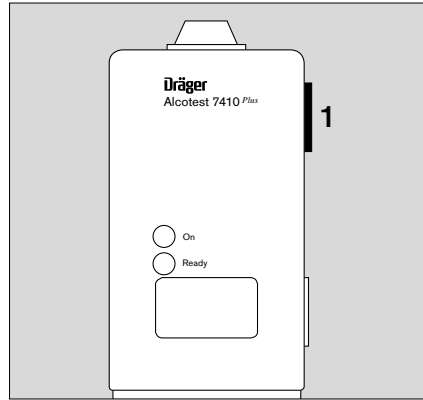
Alcotest® 7410^{Plus} RS can be connected to an IBM compatible PC via its built in RS 232-interface.

Memory options, indications of sequential test number and passive sampling can be enabled or disabled by DrägerService.

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Preparation

Alcotest® 7410^{Plus} / Alcotest® 7410^{Plus} RS is factory-calibrated.

- 1 Check calibration date on adhesive label.
- If the date is less than 6 months ago the instrument is ready for use.
 - If the date is more than 6 months ago: calibrate the unit or call DrägerService for calibration.

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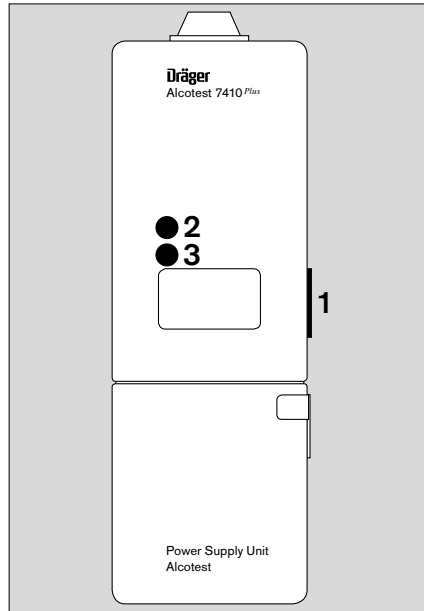
Switching on and off

Switching on

- 1 Press button once shortly.
- 2 The yellow On-light is lit and a self test including display test is carried out.
 - The symbol » **PLS** « (for "Plus") and the measuring unit are shown briefly.
 - Depending on instrument option date and time are shown:
 - » **dd** « for day, e.g. 12
 - » **nn** « for month, e.g. 08
 - » **yy** « for year, e.g. 02
 - This is followed by the display » **---** « with a full stop for the decimal point, where relevant and the symbol » **WAIT** «.
- 3 The green Ready-light is lit and a short signal tone sounds.
 - If enabled alternately the test number symbol » **no_** « or » **nr_** « is displayed followed by the current test number.

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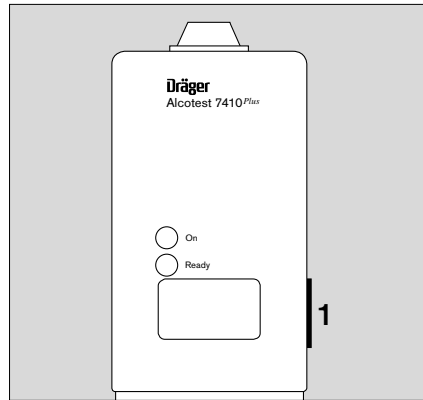
Switching off

1 Press button twice shortly directly one after the other.

When the instrument is not used after being "ready for use", it will switch off automatically 4 minutes after becoming ready for measurement.

When results are being shown on the display the instrument will switch off automatically, normally after 60 minutes.

When the power supply unit is almost flat, the instrument will switch off itself automatically.

When display indicates the battery symbol » **Lo Bat** «, the operation can be continued approx. for another 5 minutes.

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Making Measurements

General Conditions

- Use a new mouthpiece for each person being tested.
- No alcohol and solvent vapours (desinfectants) in the ambient air.
- Do not allow a high alcohol concentration to reach the sensor, as for instance from rinsing the mouth with high-percentage alcohol just before measurement, since this will shorten the lifetime of the sensor.
- Keep at a sufficient distance from the antenna of mobile phones or transmitters.
- Keep the vent at the rear of the instrument open.

Conditions for Test Person

- **There must be an interval of at least 15 minutes after alcohol has been taken into the mouth.**
Actual residues of alcohol in the mouth may give false measurements. Such residues may be left by aromatic drinks (e. g. fruit juices), alcoholic mouth sprays, medicines and drops, as well as by burping and vomiting. Rinsing out the mouth with water or non-alcoholic drinks do not substitute for an interval.
- The person being tested must breathe evenly and normally before the test. Repeated deep breaths must be avoided.

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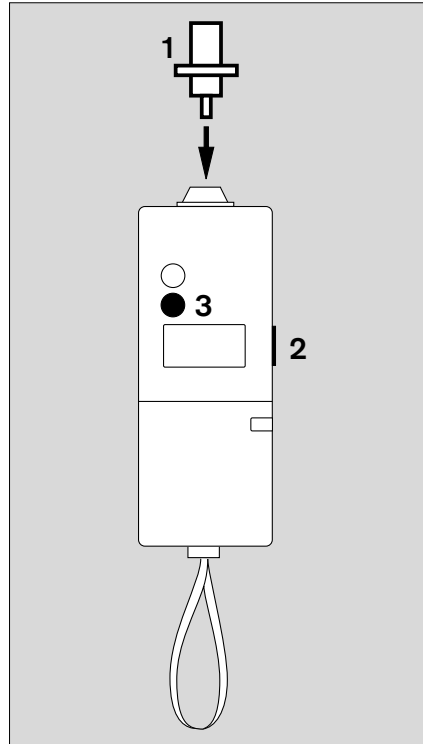
00523898_3

Measuring

- 1 Insert new mouth-piece into the holder.
- 2 Press button shortly for switching on (see page 8).
- 3 After approx. 10 to 30 seconds the green Ready-light is lit and a short signal tone sounds. The instrument is ready for measurement.

- Ask the person being tested to breathe in deeply and blow evenly into the instrument. Correct blowing is indicated by a continuous tone.
- A valid test is terminated by ceasing of green Ready-light and continuous signal tone.

For customized versions the sequence may differ.

- After several seconds the measured value will be displayed and will remain visible for normally 60 minutes or until the instrument is switched off by pressing the button twice shortly.
- After pressing the button shortly the instrument is ready for measurement again.

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If blowing into the instrument was too short or too weak,

- » E O « is displayed briefly.
- Ready-light and continuous tone cease, followed by two short signal tones, the red light is lit shortly.
- The instrument is again ready for measurement when the green Ready-light is lit.
- The person being tested must blow into the instrument again.

Interval between Two Measurements

The interval after which the instrument will be ready for a next measurement depends on the alcohol concentration previously measured (page 37, intervals before readiness for operation). The interval cannot be shortened by temporarily switching off the instrument.

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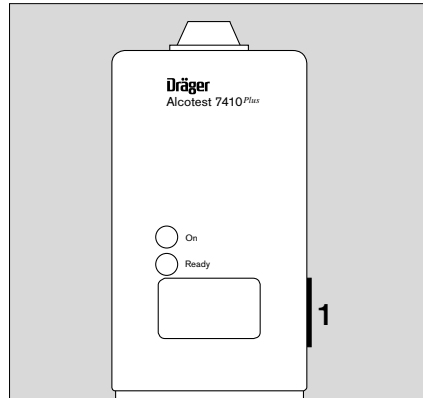
Passive Sampling

Depending on configuration the Alcotest[®] 7410^{Plus} / Alcotest[®] 7410^{Plus} RS can be used with the possibility of passive sampling.

During the passive sampling the sampling system is manually activated by the user of the instrument.

Thus if it possible to detect alcohol in the ambient air or to probe the exhaled air for the presence of alcohol of persons who are limited in the cooperation facilities (e. g. unconscious).

- Place the instrument ready for measurement without mouthpiece in the air to be analysed.
- 1 press button shortly.
 - The instrument samples the ambient air, performs by itself a measurement and analyses the alcohol content.
 - If the instrument detects no alcohol, the display shows » P «.
If alcohol was detected the display shows » F «.
- After pressing the button shortly the instrument is ready for measurement again.

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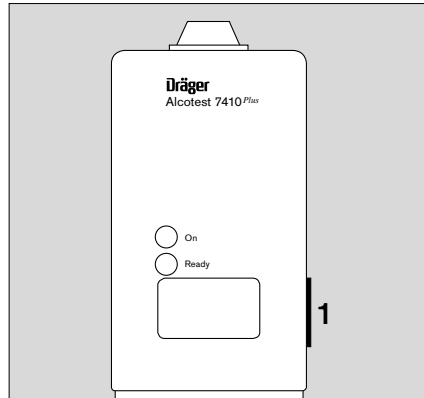
004239M_3

Memory and Data Options

Alcotest® 7410^{Plus} / Alcotest® 7410^{Plus} RS offers the possibility to show the following memory and data options:

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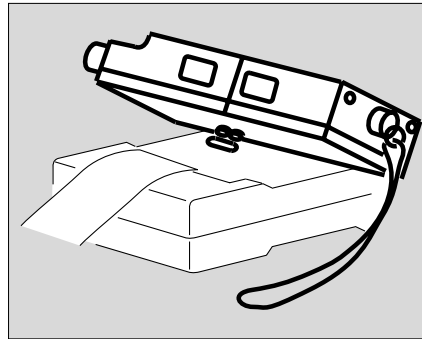
Before selection the instrument is switched off.

- 1 Press button for about 5 seconds until the display reads the first option (e.g. » **tSt** «).
- By pressing the button shortly the next option is accessible.
 - Press button for more than 2 seconds: the shown display option is chosen.

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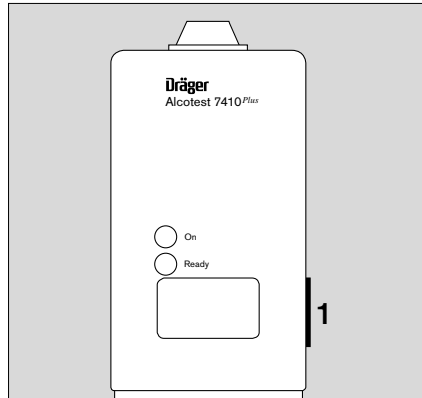
0042399_3

Option "dAt" (Data Mode)

The last measurement result of the Alcotest® 7410^{Plus} / Alcotest® 7410^{Plus} RS can be printed on the Alcotest® 7410 Printer within 60 minutes or, depending on the Printer version, indefinitely until the next test.

- When the display reads » **dAt** «, select this option by pressing the button (>2 seconds).
- 1 Press the button again (>3 seconds).
 - The display shows » **Prn** «.
 - Position the instrument onto the printer.
- Switching off:
- 1 Press button twice.

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
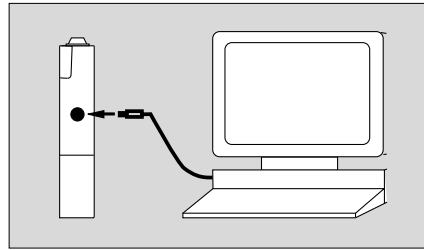
00123896_3

The built in RS 232-interface enables with Alcotest® 7410^{Plus} RS data communication with an IBM compatible PC. All data and test reports of the instrument can be read out by the software "VIEW 2000".

Additionally, data describing the reason of a measurement can be transferred for documentation into the Alcotest® 7410^{Plus} RS whereas two single measurements are required for the following measurement. The use of the software is explained in detail in the online help.

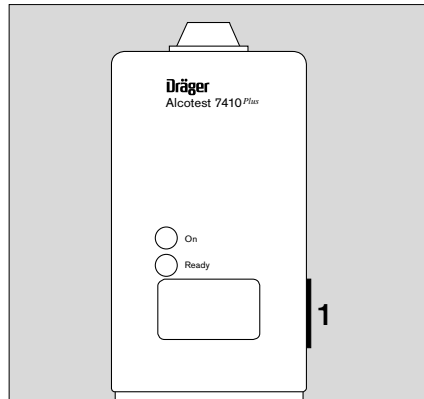- When the display reads » **dAt** «, select this option by pressing the button (>2 seconds).
- Connect Alcotest® 7410^{Plus} RS and PC (COM 1 to COM 4) by using the data transfer cable.
- Start Dräger Software "VIEW 2000" on PC. Run software option "Testdata/Load". PC displays status information. When the list of measuring results is shown, data transfer has been terminated successfully.

00423896_3

Switching off:

- 1 Press button once (>2 seconds).
- After the signal tone press button twice short-time.

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Ausschalten:

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Option "tSt" (Test)

Shows the measuring result of a particular test after selection of the test number.

- When the display reads » **tSt** «, select this option by pressing the button (>2 seconds).
- The display shows the basic setting for the test number 00001:
» **tno** « followed by » **00t** « and » **001** «.
- Press button repeatedly to select the desired flashing figure. Press longer than 2 seconds and confirm figure.
- Afterwards set the next significant figure.
- Repeat procedure for all desired figures.
- The display indicates the selected test number, the result and the measuring unit.

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2

Option "StA" (Statistics)

Shows the classification of the stored test results in different concentration ranges. The determination of the concentration ranges is done during the production of the instrument or by DrägerService.

- When the display reads » **StA** «, select this option by pressing the button (>2 seconds).
- The number of results in the determined concentration ranges is displayed one after the other.

Example:

» **ALL** «

38

thereof below a defined limit
(e. g. below 0.30 ‰, "Pass")

» **P** «

8

within a defined concentration range
(e. g. 0.30 to 0.79‰, "Alert")

» **A** «

17

above or equal to a defined limit
(e. g. 0.80‰, "Fail")

» **F** «

11

and – if defined – above or equal to a further defined limit
(e. g. 1.1‰, "High")

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The instrument switches off automatically afterwards.

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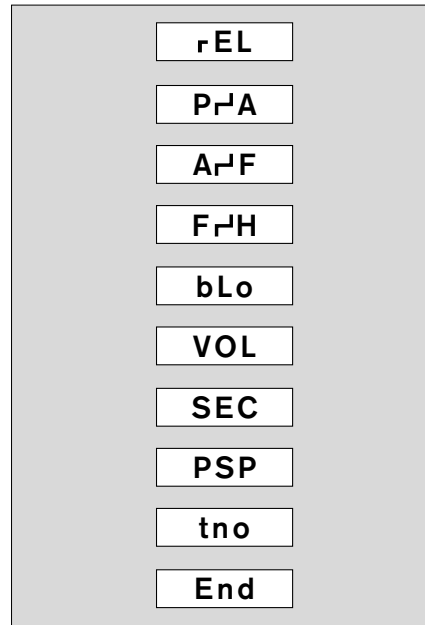
Option "con" (Configuration)

Shows the installed software version and several parameter settings.

- When the display reads » **con** «, select this option by pressing the button (>2 seconds).
- By pressing the button shortly the next parameter will be shown.

The parameters will appear as follows:

- Software release, e.g. 6.10 and 01 and 7.00
- Limit between Pass and Alert, e.g. 0.30
- Limit between Alert and Fail, e.g. 0.80
- Limit between Fail and High, e.g. 1.10
- Setting of blowing criteria, e.g. std, cl1, cl2
- Information on minimum breath volume while testing, e.g. 1,2 Litres, and measuring unit, e.g. mg/l
- Information on minimum time of blowing while testing, e.g. 4 seconds
- Activation of passive measurement, if activated, e.g. ON (activated)
- Activation of test-no. indication, e.g. ON (activated)
- End of the menu, instrument switches off automatically.

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Option "doc" (Documentation)

Offers the possibility to select the printer protocol for use with standard or extended versions of the Alcotest® 7410 Printer.

The details depend on the used options of the Alcotest® 7410^{plus} / Alcotest® 7410^{plus} RS and the Alcotest® 7410 Printer.

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Maintenance Intervals

If necessary: changing batteries, page 24 to 25.

Every six months: Check calibration, page 26 to 28.

Cleaning

- Do not submerge in liquid – liquid must not enter the sockets.

To clean:

- wipe with a disposable tissue.

Disposal of the Instrument

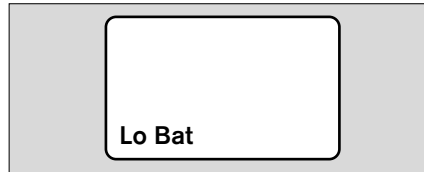
- at the end of its service life.
- Dispose of the instrument in accordance with national waste disposal regulations
or
- Ask a suitable disposal contractor to dispose of the instrument.

The local environmental agency can supply further details.

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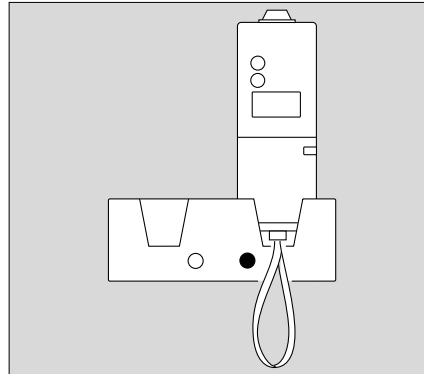
00P23199_3

Charging Power Supply Unit

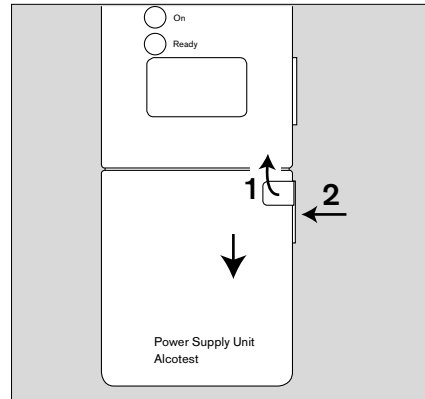
When by using a rechargeable power supply unit the » **Lo Bat** « symbol appears in the left bottom of the display or when the instrument has switched off itself before being ready for use.

- Put the instrument or rechargeable power supply unit into charger with the display to the front.
- Allow carrying strap to hang free at the front – it must not obstruct the contacts.
- Orange control lamp in charger is lit: rechargeable power supply unit is being charged. It takes 24 hours to fully recharge a completely discharged power supply unit.
- At a charging level of approx. 50 % the control lamp changes to green.
- Keep the instrument or the rechargeable power supply unit in the charger so that it is ready for immediate use.

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Replacing Power Supply Unit

- 1 Extract catch.
 - 2 Keep lock pressed in and remove power supply unit.
- Extract catch on charged power supply unit.
 - Attach measuring unit to power supply unit and push catch back into its place.

When a charged power supply unit is connected, the instrument may switch on automatically.

Used rechargeable power supply units

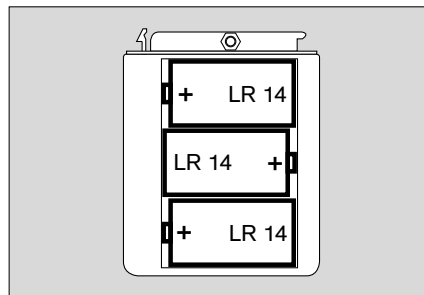
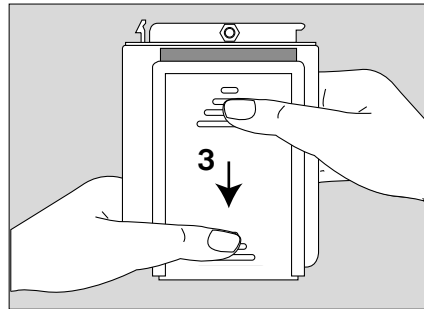
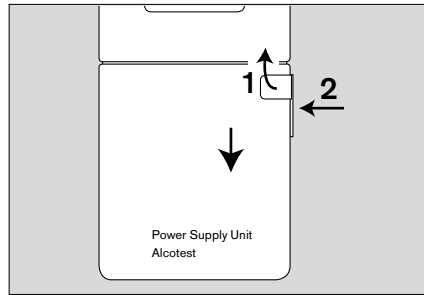
- Do not incinerate.
- Do not open violently. Risk of explosion!
- Only dispose of as special waste, according to local waste disposal regulations (contains nickel and cadmium!). Information can be obtained from environmental agencies and waste disposal companies.

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Erforderliche Batterien:
3 Stück 1,5 V (Babyzelle LR 14)

Changing Batteries

When by using an alkaline battery supply unit the » **LoBat** « Symbol appears in the left bottom of the display or when the instrument has switched off itself before being ready for use:

- 1 Extract catch.
 - 2 Keep lock pressed in and remove power supply unit.
 - 3 Shift and lift cover of the power supply unit.
- Remove used batteries and insert new ones – **consider correct polarity.**
- 3 Close battery compartment.
- Attach measuring unit to power supply unit and push catch back into its place.

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- For immediate use of the instrument after discharge of the batteries it is recommended always to have replacement batteries or an additional battery supply available.

Used alkaline batteries

- **Do not expose to fire,**
- **Do not recharge,**
- **Do not force open. Danger of an explosion!**
- **Only dispose of as special waste,** in accordance with local waste disposal regulations.
Information can be obtained from environmental agencies and from waste disposal companies.

During prolonged storage

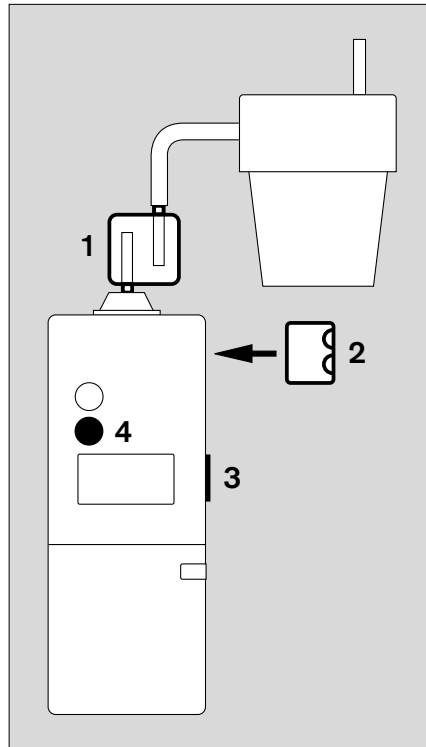
- Remove batteries from the instrument if it is not going to be used for more than 6 months, in order to prevent any damage to the instrument from leaking battery acid.

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Calibration

- Only by authorised persons.
- Note the display option, calibration range and standard calibration value.
- Keep clear the vent at the back of the instrument.
- Check calibration at least every 6 months.

With the Dräger Mark II Simulator

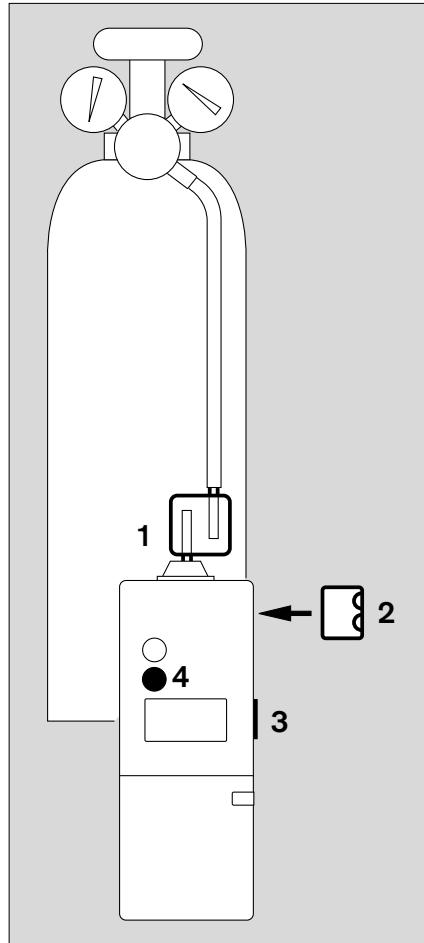
- Follow Instructions for Use for Mark II Simulator.
 - Prepare Simulator for operation.
- 1 Insert a dry adapter into mouthpiece holder and outlet hose. The hose connection should be as short as possible – not more than 5 cm – otherwise the calibration gas will cool off too quickly and condense.
 - 2 Attach calibration clip.
 - 3 Switch on the instrument. After self-test the display shows » **CAL** «. The instrument is now in the calibration mode.
 - 4 Green Ready-light is lit and a short signal tone sounds. The instrument is now ready for calibration.

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With Calibration Gas

- 1 Connect a dry adapter to the gas cylinder with a hose, not more than 1 meter long.
- Open cylinder valve briefly to flush hose and adapter with test gas.
 - Insert adapter into mouthpiece holder.
 - Continue as described in "Calibration with the Dräger Mark II-Simulator".

Note:
Take care on influence of atmospheric pressure on calibration value.

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Fault, Cause, Remedy


Fault	Cause	Remedy
Instrument switches off automatically	<p>Power supply unit / batteries discharged</p> <p>Instrument has been left ready for measurement for more than 4 minutes.</p> <p>The result of the last test has been displayed for at least 60 minutes (standard versions).</p>	Charge rechargeable power supply unit, page 22, or replace batteries, page 24
Yellow On-light is not lit after switching on or is only faintly lit; the display shows incomplete symbols	Power supply unit completely flat, or 12-Volt connection defective	Recharge rechargeable power supply unit, page 22, or replace batteries, page 24. Check or replace 12-Volt connection, page 6
Green Ready-light not lit after switching on and 4 min interval	Instrument malfunction	Call DrägerService
No continuous tone when blowing into the instrument	Instrument not yet ready for measurement	Wait until green Ready-light is lit

Fault message in display	Cause	Remedy
E 0	Test person not blowing hard or evenly enough	Ask to blow harder and more evenly
E 1	Sensor fault	Switch instrument off and on, if fault message re-appears, sensor must be replaced by DrägerService

Fault message in display	Cause	Remedy
E 2	Instrument does not recognize calibration clip	Attach calibration clip correctly
E 3	Calibration not successful Worn sensor	Calibrate again, by trained service personnel, e.g. by DrägerService Sensor must be replaced by DrägerService
E 4	Calibration value not permissible	Calibrate with permissible value
E 5	Fault in sampling system	Repeat measurement, if fault message re-appears, call DrägerService
E 6	Instrument malfunction	Call DrägerService
E 7	Data memory corrupted	Call DrägerService
FCC followed by Err red light + sound signal	Harmful electromagnetic interference was detected	Redress interference, repeat measurement.
000 followed by dAt red light + sound signal (only Alcotest® 7410 ^{Plus} RS)	Memory completely occupied, data area corrupted or not initialized	Readout of stored data. Clear memory afterwards using PC software "VIEW 2000".
-LI red light + sound signal	Low voltage of data storage battery	Readout of stored data using PC software "VIEW 2000" (only Alcotest® 7410 ^{Plus} RS). Call DrägerService.
dS red light + sound signal (only if display of time is activated)	Calibration interval expired	Calibrate the instrument, by trained service personnel, e.g. DrägerService.

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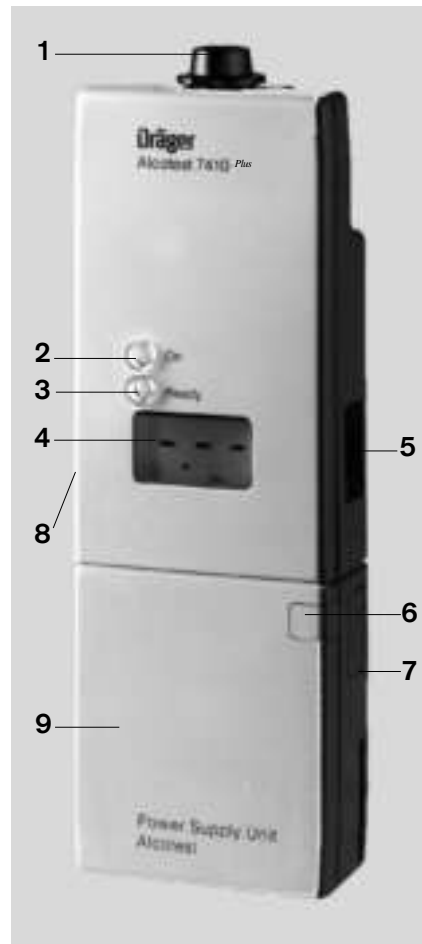Technical Data

Measuring principle Electrochemical sensor

Measurement range

Breath alcohol concentration	0.00 to 1.50 mg/L 0 to 150 µg/100 mL (Ethanol mass per breath volume at 34 °C and 1013 hPa)
Blood alcohol concentration	0.00 to 3.00 ‰ 0.00 to 3.00 g/L 0.000 to 0.300 ‰ (Ethanol mass per blood volume at 20 °C and 1013 hPa. By giving this data a conversion factor of 2100 between breath and blood alcohol concentrations is used.)

Display range

‰ Option	0.00 to 9.99 ‰
%, A % Option	0.000 to 0.999 %
mg/L Option	0.00 to 9.99 mg/L
µg/100 mL Option	0 to 999 µg/100 mL
g/L Option	0 to 9.99 g/L
P/A/F 1-Option (PAF)	
0.00 to 0.49 ‰	P (Pass)
0.50 to 0.79 ‰	A (Alert)
0.80 to 9.99 ‰	F (Fail)

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P/A/F 2-Option (PA2)
0.00 to 0.29 mg/L P (Pass)
0.30 to 0.64 mg/L A (Alert)
0.65 to 9.99 mg/L F (Fail)

P/A/F 4-Option (SAP)
0.00 to 0.21 mg/L S (Save)
0.22 to 0.34 mg/L A (Alarm)
0.35 to 9.99 mg/L P (Positiv)

Ambient conditions

For operation
-5 to 45 °C (23 to 113 °F)
-20 °C (-4 °F) for max. 30 min
and only after being kept above
0 °C (32 °F) until use.
20 to 98 % r.h.
600 to 1400 hPa

For storage
-40 to 65 °C (-40 to 149 °F),
20 to 98 % r.h.
600 to 1400 hPa

Sample conditions

Blowing time 4 to 12 s (depends on the
intensity)

Flow >6 L/min

Volume >1.2 L

Variations by options cL1 and cL2 possible.

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Measurement accuracy

Reproducibility with an
ethanol standard

0 to 1.00 ‰	±0.05 ‰
0 to 0.100 %	±0.005 %
0 to 0.50 mg/L	±0.03 mg/L
0 to 50 µg/100 mL	±3 µg/100 mL
0 to 1.00 g/L	±0.05 g/L
>1.00 ‰	±5 % of measured value
>0.100 %	
>0.50 mg/L	
>50 µg/100 mL	
>1.00 g/L	

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>100 µg/100 mL

>2,00 g/L

Intervals before readiness for operation

1st measurement about 15 s

after terminating a result of a
previous test in range:

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Calibration

Calibration interval

depends on required accuracy, typically: 6 month

Permissible calibration range for unit of measuring value:

‰	0.50 to 1.49 ‰
%	0.050 to 0.149 %
mg/L	0.24 to 0.71 mg/L
µg/100 mL	24 to 71 µg/100 mL
g/L	0.50 to 1.49 g/L

Standard calibration value when using Dräger Ethanol-Standard solution for unit of measuring value:

‰	1.00 ‰
%	0.100 %
mg/L	0.48 mg/L
µg/100 mL	48 µg/100 mL
g/L	1.00 g/L

The use of these standard calibration values corresponds to a conversion factor of 2100 between breath and blood alcohol concentration-values.

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Dimensions	70 mm x 230 mm x 34 mm (W x H x D)
Weight	approx. 500 g
Electrical supply	
Rechargeable power supply unit	3 x 1.2 V, 2.4 Ah
Charging time	24 h
Operational time at room temperature	about 400 measurements or 6 h continuous operation. With battery symbol (Lo Bat) displayed, about another 5 minutes.
Motorvehicle connection	12 V
Alkaline battery supply	3 x 1.5 V (Type LR 14)
Operational time at room temperature	approx. 600 measurements or 12 h continuous operation
Interfaces	
RS 232 interface (Alcotest® 7410 ^{Plus} RS only)	For data transmission to IBM compatible PC.
Optical interface	For data transmission to Alcotest® 7410 Printer.
CE-marking	electromagnetic compatibility (Directive 89/336/EEC)

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Order List

Name and Description	Part no.
Alcotest® 7410^{Plus} Portable measuring instrument for breath alcohol testing with data memory for about 9700 tests	83 13 630
Alcotest® 7410^{Plus} RS Portable measuring instrument for breath alcohol testing equipped with data memory for about 9700 tests and RS 232 interface for data communication with an IBM compatible PC	83 13 690
The following required when ordering: Display options: ‰, ‰, A ‰, mg/L, g/L, µg/100 mL, P/A/F, P/A/O, PA2, SAP Power supply unit: Rechargeable power supply unit, motor vehicle connection 12 V, alkaline battery supply	
Accessories	
Mouth-pieces (25 pieces) with non-return valve, hygienically tested	68 05 703
Rechargeable power supply unit	83 11 770
Alkaline battery power supply unit (for 3 x LR 14)	83 15 690
Motor vehicle connection 12 V	83 11 790
Charging station 230 V with two charging slots and duo LED	83 12 590

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Name and Description	Part no.
Alcotest® 7410 Printer	83 15 950
Software Package "RECALL"	83 13 140
For data read-out of the Alcotest® 7410 Printer and adjustment of printer options	
Leather-Case for Alcotest® 7410	78 36 774
Case for Alcotest® 7410 and Printer	83 14 210
Connecting cable for PC 25/3	83 12 160
Connecting cable with jack for 25-pole RS 232 interface on 3-pole jack plug	
Connecting cable for PC 9/3	83 13 250
Connecting cable with jack for 9-pole RS 232 interface on 3-pole jack plug	
PC-software "VIEW 2000"	83 15 140
Software for data transfer with the Alcotest® 7410 ^{Plus} RS	
Calibration accessories	
Mark II Simulator	45 40 088
Ethanol, standard solution 1.0 ‰ (1 x 500 mL)	67 28 838
Adapter (25 pieces) with water trap	68 05 473
Calibration clip	83 11 369
Calibration label (100 pieces)	68 05 739
Calibration gas, ethanol-nitrogen	on request

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Index

A ccessories	4, 40
Ambient conditions	35
Areas of explosion hazard	4
C alibration	26, 38
Calibration accessories	41
Calibration date	7
Calibration gas	28
Calibration label	27
CE marking	39
Changing batteries	24
Charging power supply unit	22
Cleaning	21
Conditions for test person	10
Configuration	19
Customized versions	11
D ata mode	15
Data options	14
Dimensions	39
Display range	34
Disposal of the instrument	21

F ehlermeldung	29, 30	D ocumentation	20
		D räger Mark II Simulator	26
G erät entsorgen	21	E lectrical supply	39
G ewicht	39	E nd	20
H aftung	5	F ault	31
		F ault message	31, 32
I nstandhaltung	4, 21	G eneral conditions	10
I nstandhaltungsintervalle	21	I ntended use	6
K alibrieraufkleber	27	I nterfaces	39
K alibrierdatum	7	I nterval between two measurements	12
K alibrieren	26	I ntervals before readiness for operation	37
K alibriergas	28	L iability	5
K alibrierung	38	M aintenance	4, 21
K alibrierzubehör	41	M aintenance intervals	21
K onfiguration	19	M aking measurement	10
M essbereich	34		
M essbereit	8		
M essen	11		
M essgenauigkeit	36		

Messprinzip	34	Measurement accuracy	36
Messung durchführen	10	Measurement range	34
		Measuring	11
		Measuring principle	34
Passive Probenahme	13	Memory options	14
Probenahme	35		
		Order list	40
Reinigen	21		
Restalkohol	10	Passive Sampling	13
		Preparation	7
Schnittstellen	39		
Sicherheit	4	Ready for measurement	8
Sonderversionen	11	Replacing power supply unit	23
Speicheroptionen	14	Residues of alcohol	10
Statistik	18		
Störung	29	Safety	4
Stromversorgung	39	Sample conditions	35
		Sensitivity drift	36
Technische Daten	34	Statistics	18
Test	17	Switching off	9
		Switching on	8
Umgebungsbedingungen	35		

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© Dräger Safety AG & Co. KGaA
Dräger (Schweiz) AG